

SmartView 5000

CONTROLS

The following controls are listed in order of their layout on the control panel from left to right.

MODE – Toggles between the three viewing modes; Photo, + Text, and – Text Modes. Photo is a full color mode for viewing images. The + Text Mode is a high contrast black text on a white background view. The – Text mode is an inverse high contrast view displaying white text on a black background.

PICTURE – This knob changes the colors used on the viewing screen. With this knob you can change the colors to a blue and white, black and yellow, red and blue, as well as a number of other color options.

SIZE – This knob controls the level of magnification used by the CCTV. Turning the knob left decreases magnification and turning it right increases magnification.

PRESET – This button causes the CCTV to alternate between two preset magnification levels. You can change a preset by selecting the preset you wish to change and after setting the magnification level you desire hold down the button for 5 seconds or until you hear a beep.

ON-OFF – This button turns the CCTV on and off. There is no need to turn on and off the monitor by itself because the CCTV controls the power going to it.

SPECIAL FUNCTIONS

FOCUS LOCK – The SmartView 5000 has an auto focus that can be annoying when flipping pages or when writing. If the Mode button is held down for 3 seconds the focus will lock. The focus setting will not change until changing the magnification level.

KEY PAD

The Key Pad plugs into the back of the neck of the SmartView 5000. This feature is not plugged in by default. If you need assistance ask a lab monitor.

Time – Displays the Time as set into the SmartView. If the button is held down for 3 seconds the display will allow you to change the time using the arrow keys in the bottom left.

Date – Displays the Weekday, Day, Month, and Year. The date can be changed by holding the DATE button down for three seconds and then using the arrow keys.

Calc – The keypad also uses the SmartView as a Calculator. This calculator only performs very basic functions like adding, subtracting, multiplying, and dividing.

Size – This button toggles between the display sizes for the date, time, and calculator.

PC – This button toggles between viewing modes for the SmartView and a computer. The Adaptive Tech Lab does not have the hardware to perform this function.

Display Line Markers – These two sets of line markers are located to the right of the PC button. The first of these sets displays two lines that can be adjusted to help a person stay on the line that they are reading when moving their text across the table. By pressing the button once the horizontal lines appear. Pressing it a second time will display the vertical lines and a third time will shut the mode off. The spacing between the horizontal lines is adjusted with the Left and Right arrow keys. Where the lines are located are adjusted by the up and down arrows. The vertical lines adjust in the opposite manner. The second key to the right of the PC button displays blank space around the text you're attempting to focus on. The options are the same as the previous button.