

Automated External Defibrillator Program

Developed by: The Department of Environmental Health and Safety

I. Purpose and Scope

This document serves to set forth responsibilities, inspections, recordkeeping and training related to the use and care of automated external defibrillators (AEDs) on the campus of Cleveland State University in accordance with applicable state and federal regulations. This document was revised in October 2019, and replaces any and all prior programs that address AED units on campus.

II. Program Administration/Medical Oversight

The CSU AED Program is managed by the Department of Environmental Health and Safety. The AED Director is the primary liaison with the Medical Director. Medical oversight is provided by the attending physician at the CSU Department of Health and Wellness Services.

Environmental Health and Safety- (216) 687-9306

University Health and Wellness Services-(216) 687-3649

III. Definitions

- (A) AED An AED (Automated External Defibrillator) is used to treat victims who experience sudden cardiac arrest and is only to be applied to victims who are unconscious, not breathing normally, and show no signs of circulation. The AED will analyze the heart rhythm and advise the operator if a shockable rhythm is detected. If a shockable rhythm is detected, the AED will charge to the appropriate energy level and advise the user to deliver a shock with voice instructions. They do not require any decision making or interpretations of symptoms.
- (B) Sudden Cardiac Arrest The unexpected and abrupt stoppage of the heart due to ventricular fibrillation.
- (C) Ventricular Fibrillation An abnormal heart rhythm often seen in sudden cardiac arrest. In this state, rhythm is chaotic and the heart cannot effectively pump blood.
- (D) CPR, Cardio Pulmonary Resuscitation The application of learned techniques in order to maintain basic life support.
- (E) Heart Attack A heart attack is a condition in which the blood supply to the heart muscle is suddenly blocked, resulting in the death of the heart muscle. Heart attack victims usually, but not always, experience chest pain and usually remain conscious. Heart attacks may sometimes lead to Sudden Cardiac Arrest.
- (F) Responder A person trained in CPR, First Aid, and use of an AED.

IV. AED Procurement

When procurement of an AED is desired, departments shall contact the Department of Environmental Health and Safety (EHS) to formally request an AED unit. EHS will facilitate acquisition of the unit with appropriate signage and ensure it is added to the master inventory of AED units on campus. The requesting department will incur the cost of the unit and its maintenance.

V. AED Inventory

EHS shall maintain an inventory of all AED units on campus that includes a serial and model number, location and date of equipment. (See Appendix A)

VI. Procedures for AED Use

- (A) Notify Campus Police Dispatch by dialing 9-1-1 from any campus phone. Cell phone users may also call 9-1-1 and when the operator answers tell them to connect you with the CSU Police. CSU dispatch can be reached directly at (216) 687-2020
- (B) Ensure all Personal Protective Equipment is present and used to address potential transmission of blood borne pathogens (barrier mask & latex gloves).
- (C) Assess the victim's condition and begin CPR and AED use based on specific training received.
- (D) Continue life saving techniques and leave the AED on until local EMS arrives, at which time they may direct you to aid them in on-going procedures or fully relieve you.
- (E) Document the use of the AED using an AED Incident Report Form and be prepared to participate in a debriefing of the incident. (See Appendix B)
- (F) Notify EHS through Campus Police Dispatch that the AED has been used. EHS will facilitate a post-incident review with involved parties, and download and store information contained on the AED unit relative to the incident.
- (G) Following AED use, the electrodes need to be replaced prior to returning AED unit to service.
- (H) Upon completion of the incident, user shall download and fill out an AED Incident Form (**See Appendix B**) and submit to the EHS Director. Copies will be forwarded to the CSU Medical Director.

If the unit cannot be returned to service immediately, post the "Temporarily out of Service" placard (**See Appendix C**) in the cabinet and notify Campus Police Dispatch at (216) 687-2020 to facilitate the needed repairs. EHS shall facilitate maintenance and pad replacement for the AED units.

VII. Training

All CSU responders must be certified in CPR and First Aid and trained in AED use. Training should conform to the American Red Cross or American Heart Association standards. Certifications in CPR and First Aid must be kept current and at a minimum, AED refresher training must be completed every two years. AED users/responders are required to maintain current certifications and provide documentation of training to EHS.

VIII. AED Maintenance and Inspection

AED units perform self-tests which automatically evaluate electronics, battery, electrodes and high voltage circuitry. Manufacturer's guidance on maintenance and inspection shall be followed. In addition, EHS will be responsible for facilitating monthly and annual inspection of the AED units. These services may be performed by CSU personnel or by an outside contracted firm.

CAUTION: Some AED's have a local alarm. Campus Safety Dispatch should be notified at (216) 687-2020 before opening an AED cabinet for maintenance and inspection purposes.

Monthly Inspection Process:

- Verification the status indicator is OK. If the indicator shows otherwise, Technical Support shall be contacted and repairs facilitated. The AED Unit should be tagged as out of service until such time repairs are made and contact to EHS should be facilitated
- Ensure a backup set of electrodes are present. If not, purchase through Manufacturer Customer Service.
- If the unit cannot be returned to service immediately, post the "Temporarily out of Service" placard (**See Appendix C**) in the cabinet and notify Campus Police Dispatch at (216) 687-2020 to facilitate the needed repairs. EHS shall facilitate maintenance and pad replacement for the AED units.

IX. Program Review

EHS shall document a review of this AED program on an annual basis and make adjustments and modifications where indicated.

Appendix A

Department of Environmental Health and Safety

Serial/Model Number	Building	Location
46940508	RC	Behind Main Desk @ Stair
3317700-000		
46940509	RC	MAC Gym Closet
3317700-000		·
46940511	RC	Fitness/Wellness Suite (RC166)
3317700-000		
46940513	RC	2nd Floor East Stair
3317700-000		
4694568 3317700-000	PE	Pool Deck
46940569		
3317700-000	PE	B60
46940570		
3317700-000	PE	Corridor Outside Room SB6
46940571	סר	Sinch Share Share to the little
3317700-000	PE	First Floor Elevator Lobby
46940572	HS	First Floor Elevator Lobby
3317700-000	113	That Floor Elevator Lobby
46940574	JH	First Floor N-S Corridor
3317700-000	311	This critical is a constant
46940505	EC 1	North Entrance
3317700-000		
46934526	EC 2	Side Entrance
3317700-000 46940617		
3317700-000	EC 3	South Entrance
46940621	EC 4	
3317700-000		Welcome Center
46940576	AC	
3317700-000		First Floor Elevator Lobby
46940614	WO	Lobby Roy Office
3317700-000		Lobby Box Office

46940615		
3317700-000	WO	Training Room
46940620	WO	
3317700-000		Practice B-Ball Gym
46940582		
3317700-000	SC	Info Desk
46940492		
3317700-000	IM	Health Services 205
46940493		
3317700-000	IM	Health Services First Floor
46940583		Second Floor Elevator Lobby
3317700-000	LL	
43795238	110	
3200731-010	UR	First Floor Elevator Lobby
43795241	UR	Second Floor Elevator Lobby
3200731-010	Oit	Second Floor Elevator Lobby
43795245	UR	Third Floor Elevator Lobby
3200731-010		,
46940502	CS	Lobby
3317700-000		,
46940503 3317700-000	CS	Mobile Unit - Bag # 1
46940506		
3317700-000	CS	Mobile Unit - Bag #2
46940504		
3317700-000	CS	Mobile Unit - Bag #3
46940507		
3317700-000	CS	Mobile Unit - Bag #4
43498591	1/5::	
3200731-010	KFH	Athletic Training Room
43795390	KFH	Athletic Training Room
3200731-010		·
43998586	Tennis	By main
3200731-010	Courts	entrance
45832789	SPARE SPARE	PS 234
3317700-000		
47271981		PS 234
3317700-000	RT	
46086736		Library
3317700-000		
45618644 3200731-009	RT	10th Floor
3200/31-009		

	46355458	DT	11+b Floor
3317700-000	KI	11th Floor	

Appendix B

Cleveland State University Department of Environmental Health and Safety

AED Incident Form

AED Users: Every incident where an AED Unit is used shall be documented by completing this form in its entirety and submitting to EHS within twenty-four (24) hours of the incident.

Name of Patient	Phone Number			
Male/Female (Circle One) Date of Bi	irth Age			
Date/Time of AED Use	_ AED Serial Number:			
Location of Incident				
Description of Incident:				
Were any witnesses present? Yes	No			
If yes, provide contact information				
Identify EMS Unit that responds/transports				
Time and destination (if available) of	transport			
Name of AED Operator	Phone			
AED Operator's Signature	Date			

Appendix C



AED Notice

AED Temporarily Out of Service

Scheduled Maintenance
Repair
Missing
Other
Additional Details:

Affix this Notice to the AED Unit and report to CSU Police Dispatch (216) 687-2020